Standing slings

*Standing
Standing 2-belts
Fleece*

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Date: 16.12.2013 - ver. 4

EN: Standing slings

Important.....	4
Placement of sling straps.....	5
Lifting.....	6
Standing.....	7
Standing 2-belts.....	7
Fleece.....	7

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Important

This manual must be read before the sling is used. This manual is intended for therapists and other personnel, in connection with the use and maintenance of *Ergo* slings.

Purpose

Ergolet's *Ergo* slings can be used for transfers in home care, nursing homes, institutions and private homes. *Ergo* slings are used to transfer people with physical or reduced mobility issues and for transfers from bed to chair, from chair to bathing and toilet chair etc.

Check list before use

It is the carer's responsibility to check the sling before use.

- It must be ensured that the sling does not have any visible damages or wear and tear and that the sling is clean.
- When the sling is used, qualified personnel must be present.
- The capacity written on the sling must never be exceeded!

Lifetime

The sling can be washed 80-100 times, which corresponds to once a month in 6-8 years. The product's lifetime is expected to be 5-10 years, if service checks and the stated maintenance instructions have been followed. It is the owner's responsibility that the annual inspection is performed.

Materials

The slings are produced in durable materials and tolerate the necessary washing. All slings are produced in polyester.

Label symbols

Declaration of Conformity

Recommended 80°C for washing

Do not bleach

Do not iron

Do not dry clean

Tumble dry at low temperature

Max 250 kg

Maximum lifting capacity is 250 kg

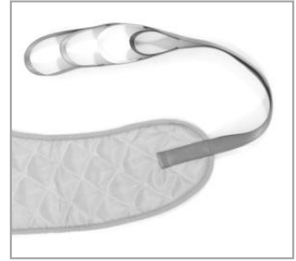
Recommended 92°C for washing
(Fleece sling)

Placement of sling straps

Colour coded straps

The straps on most of the slings are marked with colours (red, yellow, green, blue), which makes adjustment of position and placement of the sling easier.

Corresponding sling straps must be attached to the spreader bar with the same colour. The sitting position can be varied by choosing different colours for the leg and shoulder straps.

Patented Ergolet sling hook system

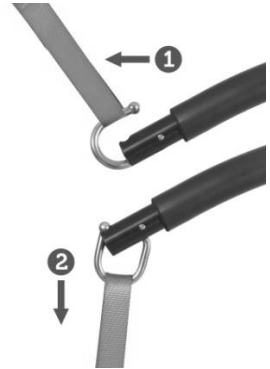
Ergolet does not take any responsibility if the sling is placed incorrectly. It is always the carer's responsibility that the correct sling has been chosen and the sling is fitted correctly.

1. The sling strap is placed on the open hook (1).
2. The strap is pulled downwards and the hook will close (2).

The design of the lifting hook is such that the weight of the user will help close the hook.


To release the strap after lifting simply pull upwards to open the hook.

Note: Ensure that client's fingers cannot get trapped.


Other hooks

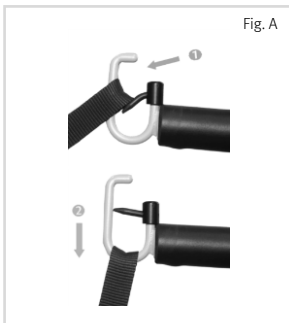The fig A below shows placement of the strap on hooks of Pallas/150 stand aid.

Strap color code

RED	SMALL
YELLOW	MEDIUM
GREEN	LARGE
BLUE	X-LARGE

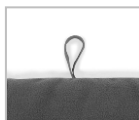Lifting

It is important to know to what degree the user can actively assist during the lifting.

This sling can be used with seat sling (Fig 3) for extra support, when using Ergolet's stand aid Pallas.

1. Place sling around waist.
2. Tighten the safety belt (Fig. 1) while making sure that the standing sling has the correct size for the user.
3. Attach the straps to the hooks on the lifter using the same colour straps on both sides (Fig. 2).
4. Start lifting and tighten the belt again.

- Ensure that sling loops are placed correctly on the spreader bar hooks.
- Ensure the patient sits/lies safely before taking off the sling.
- Standing slings must only be used with a stand aid.

Above: A lift in the Ergolet Standing sling (with 2 belts) onto the Pallas/150 stand aid, from Hera/140 toileting chair.

Fig. 3

Fig. 1

Fig. 2

Standing slings

Standing sling with 2 belts

USABILITY

Suitable for stand aids and for gait training with a stand aid.

For extra comfort, the Fleece Sling has no sharp edges and is extra soft. For users with a big stomach, the 2-belt sling is suitable.

Models

Standing Sling


Standing Sling w. 2 belts

Fleece Standing Sling


TECHNICAL SPECIFICATION

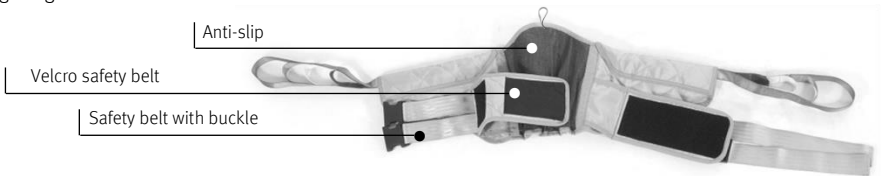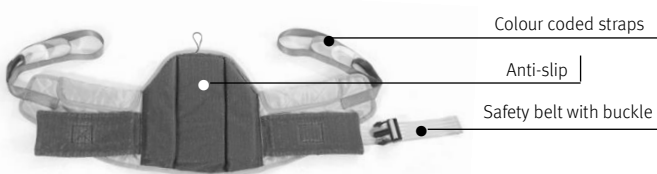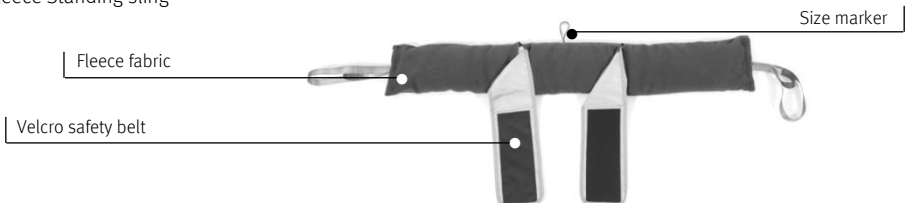
Max capacity	250 kg
Max washing temperature	80°C, (Fleece: 92°C)
Max drying temperature	55°C
Attachment	Hooks on lifting arm
Sling is available in sizes	XS - XXL; Fleece: S - XXL

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