WINN CARE

 **Site de production**
Miniac-Morvan (35)

 IFU_WINNSAVE

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1. INDICATIONS

Device Intended Use

Prevention of the risk of falls and/or disability compensation.

Indications

WINN'SAVE's restraint or support systems require a medical prescription. These products are designed and used to protect the patient themselves or their environment.

- › Physical restraint: to restrict freedom of movement in a patient who is agitated and/or aggressive with cognitive or psychological issues, to prevent the risk of falls and protect staff or the patient themselves.
- › Support: to protect a confused patient from getting up when not medically authorised or to enable a treatment procedure to be carried out.

Contraindications

Patients who do not fit the size of the device.

Target patient and user group

Adults who are hospitalized, institutionalized or at home, at risk of falls. These individuals are cared for by health professionals, with the help of assistants if required.

Indicate undesirable side effects

Any serious incident related to the device should be notified to the manufacturer and the competent authority of the Member State where the user and/or the patient is established.

Inform the competent authority if you consider or have reason to believe that the device presents a serious risk or that it is a fake product.

2. COMPOSITION OF THE MEDICAL DEVICE

Restraint or support device with closure system (self-fastening straps or buckles),

Identification label present on the product,
Instructions for use (paper and/or electronic).

3. CLINICAL BENEFIT, PERFORMANCE

Performance characteristics of the device

Medical devices for postural support or restraint help secure the patient and limit risks of falls (from the bed or wheelchair).

Expected clinical benefits

Restraint devices

- › In bed, limit the risk of the patient falling, reduce risks related to wandering, facilitate treatment procedures or immobilize the patient post-surgery.
- › In the wheelchair, limit the risk of falling, reduce risks related to wandering or immobilize the patient after surgery or a fracture.

Postural support devices

- › In bed, limit the risk of movement of a limb during treatment or a medical procedure, reduce the risk of movement during an exam or after surgery.
- › In the chair, limit the risk of the torso tilting or the body slipping in the sagittal plane, or removal of a probe or catheter.

4. INSTRUCTIONS FOR USE

User training

We recommend that healthcare personnel be trained in use of these medical devices and be aware of the need to monitor patients during use to reduce incidents of materiovigilance

Device installation

PELVIC SUPPORT

Pelvic strap

- › Place the strap on the wheelchair, close and adjust the strap at the back of the wheelchair.
- › Sit the patient down and move them as far back as possible.
- › Pass the pelvic straps through the legs and fold them onto the legs.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Pelvic breeches

Classic

- › Place the narrowest part of the breeche on the seat and attach it behind the wheelchair .
- › Sit the patient down.
- › Pass the widest part between the legs and attach the fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Ergostyl

- › Attach the breeche at the rear of the wheelchair.
- › Sit the patient down.
- › Pass the front of the breeche between the legs and attach the fastener behind the wheelchair.
- › Adjust the belts to fit the patient's shape and size.

Pelvic belt

Classic

- › Unfasten the self-fastening straps.
- › Place the belt around the patient and the wheelchair.
- › Attach the self-fastening straps behind the wheelchair and adjust according to the patient's shape and size.
- › Pass the strap between the legs and under the wheelchair, and attach it to the belt behind the wheelchair.

Ergostyl

- › Unfasten the buckles and place the belt around the patient.
- › Pass the pelvic strap between the legs and sit the patient down.
- › Pass the belt strap in the strap buckle between the legs.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

ABDOMINAL SUPPORT

Belly belts

Classic

- › Unfasten the self-fastening straps.
- › Place the belt around the patient and the wheelchair.
- › Attach the self-fastening straps behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Classic

- › Unfasten the buckle and the quick fastener.
- › Place the belt around the patient.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Comfort

- › Place the belt around the patient.
- › Pass the strap through the belt opening and pass the straps behind the wheelchair.
- › Pass the strap through the tightening buckle and adjust it to fit the patient's shape and size.

BUST SUPPORT

Support jacket

Classic

- › Place the jacket on the patient and sit them down.
- › Pass the free strap through the holes on the shoulders and attach it behind the wheelchair.
- › Pass the strap at the bottom of the jacket through the opening, adjust to fit the patient's shape and size behind the wheelchair.

Ergostyl

- › Unfasten the buckle, place the jacket on the patient and sit them down.
- › Attach the quick fastener behind the wheelchair.
- › Pass the free strap through the shoulder rings and attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Comfort

- › Unfasten the buckle, place the jacket on the patient and sit them down.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Safety bra

- › Unfasten the buckle and the quick fastener.
- › Place the belt around the patient, then put on the vest.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Classic

Harness

- › Sit the patient down on the wheelchair.
- › Open the harness using the clip.
- › Position the harness with the crossover part in front, the end of the straps wrapped around the wheelchair handles.
- › Attach the belt after carefully passing it under the armrests.
- › Adjust the tension and position of the belt using the strap tie.
- › Adjust the tension of the shoulder straps and position the neoprene on the shoulders.

INTEGRAL BODY SUPPORT

Pelvic bra

- › Attach the seat part behind the wheelchair and sit the patient down.
- › Pass the front of the seat part between the legs and put on the upper part.
- › Attach the quick fastener behind the wheelchair.
- › Adjust the straps to fit the patient's shape and size.

Harness

Ergostyl

- › Unfasten all the buckles to obtain 2 parts: the belt and the harness.
- › Place the belt on the wheelchair, pass the strap through the Y-shaped opening then close and adjust the strap at the back of the wheelchair.
- › Sit the patient down on the wheelchair, making sure they are as far back as possible.
- › Place the harness on the patient, letting the coloured buckles hang behind.
- › Pass the pelvic straps through the legs and fold them onto the legs.
- › Pass the straps through the belt openings then through the harness openings.
- › Behind the wheelchair, close the coloured buckles.
- › Adjust the straps to fit the patient's shape and size.

Comfort

- › Unfasten the buckles and place the belt around the patient.
- › Pass the pelvic strap between the legs and sit the patient down.
- › Attach the belt's quick fastener behind the wheelchair.
- › Pass the harness straps over the shoulders.
- › Attach the quick fasteners between the harness and the pelvic strap behind the wheelchair.
- › Place the protective pads on the shoulders.
- › Adjust the straps to fit the patient's shape and size.

BED SUPPORT

Bed belts

Classic

- › Unfasten the buckle and the quick fastener.
- › Place the belt around the patient.
- › Attach the quick fastener behind the bed.
- › Adjust the straps to fit the patient's shape and size.

Ergostyl

- › Place the belt flat on the bed with the abdominal belt on top.
- › Attach the straps on each side to the bed post using the quick fastener.
- › Adjust the abdominal belt around the patient and attach it using the self-fastening straps.
- › Patient on their side: Place the patient on their side, tighten the two fabric straps using the two tightening buckles at the back. The patient may move freely in the sideways direction.
- › Patient on their back: Adjust the side straps on each side of the gripper located near the bed post. The patient may no longer move in the sideways direction.

Immobilisation gutters

- › Lace the arm or leg onto the gutter and attach the self-fastening straps to fit the patient's shape and size.

SUPPORT ACCESSORIES

Arm slings

Classic

- › Place the arm to be immobilized in the sleeve. Push the elbow to the bottom of the sleeve and place the thumb in the ring.
- › Pass the strap over the shoulder and close the quick fastener.
- › Adjust the straps to fit the patient's shape and size and position the neoprene strap on the shoulder.

Ergostyl

- › Place the arm to be immobilized in the sleeve and attach the self-fastening straps.
- › Pass the strap over the shoulder and close the quick fastener.
- › Adjust the strap to fit the patient's shape and size.

Stop Gliss

- › Place the product directly on the wheelchair seat or on the cushion.
- › Pay attention to the right direction: "Slide direction" arrow pointing towards the back of the wheelchair and "no slide" towards the front.

Security mitts

- › Place the glove on the patient's hand.
- › Adjust the tension of the strap, close the clip on the top of the wrist.
- › Reposition the strap tie on the self-fastening straps.

Wrist ties

Classic

- › Wrap the fabric straps around the wrist and attach the self-fastening straps.

- › Pass the strap around the armrest and close the quick fastener.
- › Adjust the strap to fit the patient's shape and size.

Ergostyl

- › Wrap the neoprene around the wrist.
- › Close the small buckle and adjust the small strap.
- › Close the large strap around the support, making sure that it is around the support and the wrist.

Preventive maintenance

- › Regularly check restraint devices in use and in stock for physical restraint. They must be clean, complete, in good condition and stored such that they can be used quickly.
- › Prohibit any modifications or repairs to restraint and support devices.

Cleaning and disinfection

- › Between each patient.
- › Machine wash up to 40°C.
- › Do not iron.
- › Use bleach-free detergents.

5. WARNINGS AND PRECAUTIONS FOR USE

Precautions for use

- › This product must be used by staff trained in accordance with the manufacturer's instructions, in line with safety procedures and rules from recommendation of best practices applicable in local authorities and home care.
- › Before use, check that the device's characteristics are intact.
- › Use of these devices requires a medical prescription following multidisciplinary assessment.
- › The prescription specifies the type of product, the duration of use (continuous or occasional) and the frequency of reassessment. By default, daily reassessment of the benefit/risk ratio is recommended.
- › When using with a patient requiring additional precautionary measures, the device may be cleaned and disinfected using a protocol suited to the risk of infection or disposed of.
- › Use only the closures provided or approved by the manufacturer.
- › Check that agitated patients cannot access and handle closure systems.
- › Attach the self-fastening straps to each other, not to the fabric.

Warnings

- › Any installation of a medical restraint or support device shall require observation of the patient's behaviour beforehand. This upstream observation must be included in the patient's medical file.
- › When using a therapeutic motorized air mattress to prevent bed sores, we recommend that you adjust the length of the attachments/straps attaching the medical restraint device to the bed, by placing the mattress in maximum inflation mode (do not adjust the length of the attachments/straps when the mattress is in alternate or low pressure static mode). Make sure that the patient is not compressed when inflating the mattress or trapped between the structural elements of the side barriers, and conversely, that the restraint straps do not become too loose when the mattress is deflated.
- › Store unused devices out of the reach of patients.

Lifetime information

The device may be reused within the initial performance maintenance limits during its life cycle. Its estimated lifespan is 4 years.

Spare parts

Only closure buckles supplied by the manufacturer can ensure the safety and performance of the complete product.

6. STORAGE/ HANDLING/ DISPOSAL

Special storage and/or handling conditions

- › Store away from light, heat, and humidity.

Disposal of the product

Do not throw the product out into non-dedicated locations in nature. Follow the recycling channels that are in place in your country.

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