

QUICKGUIDE TRITON/DUAL

Shower- and toilet chair (max user weight 150 kg)

UP —▲

DOWN —▼

TRANSPORT — Positions the chair for safe transfer of the user

SAVE —S
ADJUST the chair to the desired position.
PRESS DOWN S + 1 or 2 for two seconds.
 When you hear a **BEEP**, the desired position is saved in memory button 1 or 2.

TILT FORWARD
 Adjusts the seat tilt up to +7°

TILT BACKWARDS
 Adjusts the seat tilt down to 30°

1 — **MEMORY 1**
 See instructions under 'SAVE'

2 — **MEMORY 2**
 See instructions under 'SAVE'

TRITON/DUAL

WINCARE
N O R D I C



TIP!
 Use the chair's memory functions for quick and precise positioning, e.g. for hair wash or toileting.

EMERGENCY STOP & LOWERING (add-on)

A factory mounted emergency stop and emergency lowering is available as an add-on, if desired.

● Emergency stop

Push to lock.
 Turn to reset.

● Emergency lowering

Can be used to lower the chair, if for example the hand control is defect.



BATTERY

Up to 100 lifting cycles per charge.
 Must be charged regularly, e.g. over night.

Battery low voltage buzzer.

LED-INDICATOR

Colour Indicates

- Yellow: Charging
- No light: Fully charged

